

ATOMO DIAGNOSTICS

atomo HIV SELF TEST



Atomo offers the world's first integrated blood-based rapid diagnostic test (RDT) for HIV screening.

Unlike clinical testing, The Atomo HIV Self Test provides results in just 15 minutes. The Atomo HIV Self Test is highly accurate, with a specificity and sensitivity level of 99.6% due to interlocked user steps that ensure the test can be comfortably carried out by lay users in a private and discreet setting.

FREQUENTLY ASKED QUESTIONS

What is the Atomo HIV Self Test?

The Atomo HIV Self Test is a single-use blood test for HIV. It is comprised of a test strip inside a plastic cartridge. The test is performed by placing a small drop of blood onto the test strip and then applying drops of test fluid (diluent). One or two lines will appear on the paper strip after 15 minutes to indicate the

test result. The Control Line will only become visible if the test has been performed correctly. The Test Line will only become visible if the applied blood sample contains antibodies to HIV.

How does the Atomo HIV Self Test detect HIV?

There are 2 types of Human Immunodeficiency Virus (HIV):

HIV-1 and HIV-2. If you are infected with either HIV-1 or HIV-2 your immune system will generate antibodies against either virus type. The test detects antibodies in human blood to determine infection by either virus type.

How much is RRP?

RRP is \$29.99



>99.6% Accurate



15 Minute Results



Test At Home

Level 1, 3-5 George St.
Leichhardt, NSW, 2040
(0) 2 9099 4750



www.atomodiagnosics.com
sales@atomodiagnosics.com

ATOMO DIAGNOSTICS

Can a customer use the test if they are taking PrEP?

Yes, people on PrEP can use the test. Adherence to the window period recommendations is required.

How many times can the device be used?

The Atomo HIV Self Test is a single-use test. It can only be used once and must be disposed after use.

How is the test disposed after use?

Place the bag and all contents into the disposal bag provided.

The bag can be sealed and thrown away with household rubbish.

How will I know if my test has worked?

If you see a line next to the 'C' symbol on the test cassette, it means you have performed the test correctly.

What happens if my test result is REACTIVE?

The Atomo HIV Self Test is a screening test and a reactive test result does not necessarily mean that you have HIV. If the test result is reactive, you need to go for follow-up testing at a healthcare facility. Visit www.atomohivtest.com for more information about accessing follow-up testing.

If you are diagnosed as HIV positive, you will be connected to counselling services and, dependent on your condition, may be given antiretroviral (ARV) treatment. Only your doctor or healthcare professional can recommend what treatment is right for you.

Early diagnosis of HIV means treatment can start sooner. HIV is a manageable condition.

What is the window period?


After exposure to HIV, it can be 3 months before your body develops the antibodies detected by this test. This time is called the 'window period'. If you take an HIV test during the window period, you may incorrectly receive a negative result. It is essential to test after the window period has passed.

Does the test have regulatory approval?

Yes, the self-test is listed on the Australian Register of Therapeutic Goods(311989) since 2018.

Is there a separate lancet in case the mechanism doesn't work?

No separate lancet is included in the pack. Reported incidents of lancet failure are uncommon.

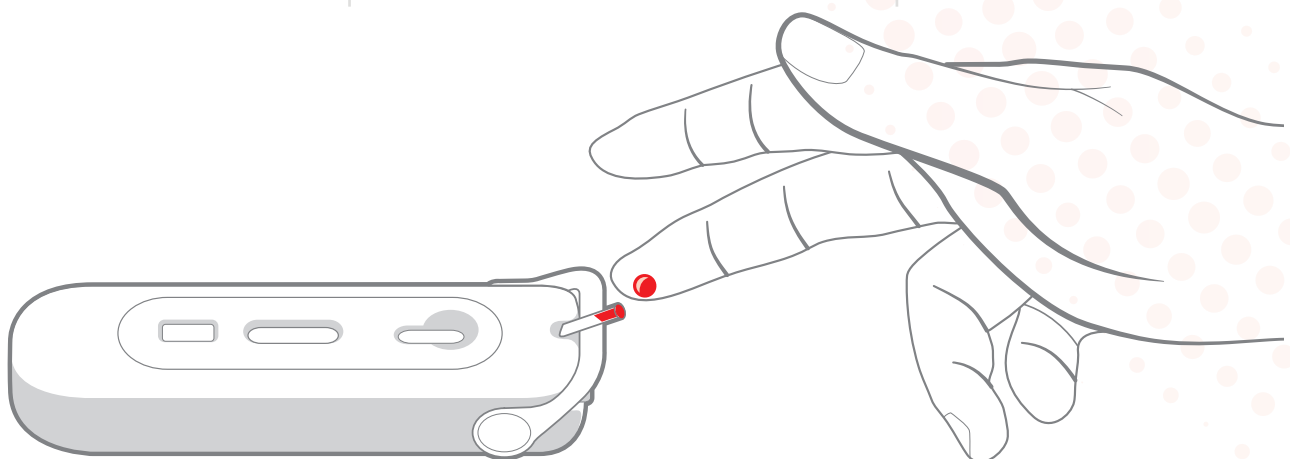

LBL-106-01